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EN-GB: APPLICATION & USE: The earplugs provide protection against the harmful effects of noise. It is recommended that you strictly follow all instructions of the manufacturer; otherwise the protective effect may be seriously impaired. Do not insert the earplugs too deeply into the ear. Before use, check whether the earplugs are clean and undamaged. Grab the earplugs with thumbs and index fingers and insert them completely by slightly rotating them until the lamellae completely close the ear canal. The grip of the earplug should be facing inward towards the head when inserted into the ear. In noisy areas, the hearing-protection earplugs must be worn all the time. Furthermore, it is recommended not to modify the product and to exactly follow the instructions for use, storage and cleaning. The earplugs may only be inserted into the ears as intended for protection against noise and not into other body orifices. The product should be replaced in 10/01/2023 at the latest. **REMOVAL:** Pull the tab by slowly rotating it. After removal, store the earplugs in the included aluminum container. **CLEANING:** When necessary, clean with a damp cloth or mild detergent (soap). Do not clean with chemicals or alcohol, as this

can reduce the protective effect. **WARNING:** Sudden or quick removal of the earplug may cause damage to the eardrum. Rooth C&P is not liable for any damage arising from misuse.

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